

- 1) Place the hardboard in the middle of the crib between the base of the crib and the mattress.
- 2) Place the SensorPad – print side up – on top of the hardboard in the middle of the crib.
- 3) Safely pass the SensorPad cord through the slatted base/springs of the crib keeping the cord **well** away from baby.

IMPORTANT NOTICE:

If your crib has a solid hardwood base; drill a hole in the hardwood base and pass the SensorPad cord safely through the hole, keeping the cord **well** away from baby.


- 4) Safely secure the SensorPad cord by wrapping it around one of the crib legs and attach it securely using a plastic cable tie.
- 5) Plug the SensorPad cord into the Nursery Unit, with the SensorPad cord running safely **well out of reach of children**.
- 6) The Nursery Unit and Parent Unit **MUST** be a minimum of **1m (3ft)** away from the cot.

• The SensorPad functions with any size and type of mattress except memory-foam mattresses and hollow-frame mattresses.

• The hardboard maintains a constant pressure on the SensorPad, enabling it to detect the baby's movements even when the baby is at the edges of the crib.

STRANGULATION HAZARD

Children have STRANGLED in cords. Keep the Adapter cords and SensorPad cord out of the reach of children. DO NOT put the Nursery Unit inside a crib or play yard. Because of the serious strangulation risk to infants and toddlers, parents and caregivers should never place the Nursery Unit within 1 meter (3 feet) of a crib. NEVER use extension cords with AC Adapters. ONLY use the AC Adapters provided.

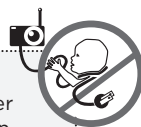

IMPORTANT

- Keep the cord on the floor, **well out of reach of children**.
- The SensorPad is not designed for use with adults, children or infants older than 24 months.
- Once you have finished using the SensorPad, please remove it from the baby's crib and store it away.

IMPORTANT INFORMATION

Dear Angelcare user,

For over 15 years Angelcare has been dedicated to providing high quality, safe products that offer peace of mind to families all over the world. It is and will always be Angelcare's mission to provide the best product to families everywhere.

To register for a free sensor pad cord cover kit, please visit our dedicated website:

www.angelcarekit.co.uk

If registered correctly, you will receive a confirmation email within an hour.

Even without this cord cover, your Angelcare movement monitor is safe to use, if installed correctly according to the UK instructions.

Please find instructions on how to install your SensorPad correctly without the cord cover on the reverse of this leaflet.

All Angelcare monitors should continue to be used, no hazard exists when the monitor cords are kept out of reach of your child.

Requests for a kit through our Baby Helpline email address cannot be accepted and will not be processed. Please use our dedicated website only for your kit request.

If you have any further questions relating to correct installation of your monitor or about the cord cover kit, please contact us.

Angelcare UK Helpline:

0845 009 1789

(Charges apply)

email: helpline@babyhelpline.co.uk

We would like to take this opportunity to personally thank all our loyal customers. As parents ourselves, our mission will always be to deliver the best products to families in the UK.

Yours sincerely,

Angelcare UK

Angelcare®